**EU Master Text  
INSTRUCTIONS FOR USE  
Lotrafilcon A Soft Contact Lenses (Spherical)**

| Product TN                                    | Lens Design |
|---|-------------|
| AIR OPTIX™ NIGHT & DAY™ AQUA<br>AIR OPTIX™ EX | Spherical   |

With handling tint (TN: VISITINT™)

Note(s): Above or future ALCON approved/trademarked names (TN) and/or descriptors may be used. Text appearing in gray italics font and between < > symbols is meant for reader instructions and is not intended to appear in actual labeling components, except when options in black font are being provided.

| <b>Document Revision History</b> |  |
|----------------------------------|--|
| <b>Version</b>                   | <b>Description</b>   |
| DEC 2015                         | Multi-Country EU cSLT (IFU only)   |
| SEP 2018                         | Original issue EU Master Text (IFU) under revised CORP-002739 v.5.   |
| JUL 2019                         | Revise <b>Serious Incident Reporting</b> ; clarify <b>Performance Characteristics</b> ; add <b>Intended Purpose/Use, Patient Target Groups, and Clinical Benefits</b> ; add requirements for ISO 11978; revise <b>Contraindications</b> ; revise <b>Warnings</b> ; revise <b>Precautions</b> ; revise <b>Directions for Use</b> ; revise recommended <b>Lens Care Solutions</b> ; add <b>Disposal and Recycling</b> ; revise <b>Contents of Container</b> ; revise glossary table (add symbols); revise EC REP address; revise copyright and trademarks; add guidance text for Private Label IFU |

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- AOSEPT™ PLUS with HydraGlyde™ Cleaning and Disinfecting Solution
- OPTI-FREE™ PureMoist™ Multi-Purpose Disinfecting Solution

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## REFERENCES

Cutter GR, Chalmers RL, Roseman M. The Clinical Presentation, Prevalence, and Risk Factors of Focal Corneal Infiltrates in Soft Contact Lens Wearers. The CLAO Journal. Jan 1996; 22 (1): 30-37.

Schein OD, Glynn RJ, Poggio EC, Seddon JM, Kenyon KR. The Relative Risk of Ulcerative Keratitis Among Users of Daily-Wear and Extended-Wear Soft Contact Lenses. N Eng J Med. 1989; 321(12):773-83.

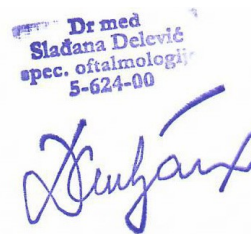
## SKRAĆENICE I SIMBOLI KOJI SE MOGU KORISTITI ZA OBELEŽAVANJE

| SKRAĆENICA / SIMBOL   | DEFINICIJA                                |
|---|---|
| BC  | Bazna krivina                             |
| DIA   | Promer                                    |
| PWR   | Snaga                                     |
| D   | Dioptrija (sočiva)                        |
| L   | Levo                                      |
| R   | Desno                                     |
| VP/DMAEMA   | Vinilpirolidon/dimetilaminoetilmetakrilat |
|  | Oznaka ambalažnog otpada                  |
|  | Serija                                    |
|  | Iskoristi do                              |
| EXP   | Rok upotrebe (Iskoristiti do)             |
|  | Jedan sistem sterilne obloge              |
|  | Sterilisano parom                         |

|   |  |
|---|--|
|  | CE oznaka  |
|  | Jezična oznaka (u primeru: Engleski)   |
|  | Oprez  |
|  | Pročitajte uputstvo  |
|  | Ne koristiti ukoliko je blister pakovanje oštećeno   |
|  | Proizvođač   |
|  | Datum proizvodnje  |
|  | Medicinsko sredstvo  |
|  | Ovlašćeni predstavnik u Europskoj Uniji  |
|  | Oprez: Federalni (SAD) zakon ograničava ovaj proizvod na prodaju od strane ili po zahtevu očnog specijaliste |

  
 Manufacturer  
 Alcon Laboratories, Inc.  
 6201 South Freeway  
 Fort Worth, TX 76134-2099, SAD

  
 Alcon Laboratories Belgium  
 Lichterveld 3  
 2870 Puurs-Sint-Amands  
 Belgija

  
 Dr med  
 Slađana Delević  
 spec. oftalmologije  
 5-624-00

CE 0123 < symbol >

Alcon < logo > < remove for private label >

< Insert labeling component part number >

Ovlašćeni predstavnik i distributer: OPTICUS d.o.o., Kneza Mihaila 6, 11000 Beograd, Republika Srbija  
 Broj rešenja:

**Datum izdavanja:** < Insert month and year of document, with Revision identifier specified (e.g., JUL 2019, Rev. X >

© < Year > Alcon Inc. < remove for private label >